

MOUNTING REFERENCE GUIDE

Keyboard & Mouse


CIM Cable Integrated Mounting System





Content

General notes	3
How to create part numbers	4
Product highlights	6
Support arm pivot	7
Extended pivot arm light duty	9
Height adjustable arm	11
Extended height adjustable arm	13
Keyboard tray options	15
Fixings	19

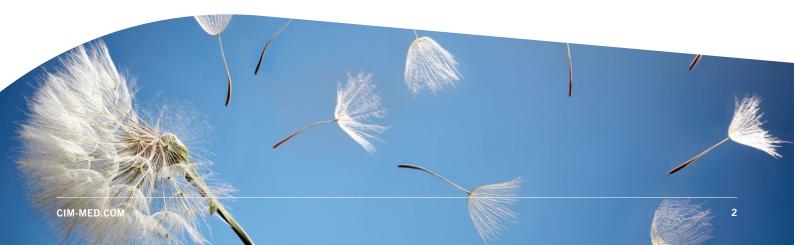





Components' grounding

All arms with integrated cable management are equipped with **component grounding**. No grounding is required for arms with external or semi-integrated cable management with cables not being led through joints.

Load capacity

All height adjustable arms have a load capacity of 22 kg / 48.5 lbs. Rear mounted monitor adaptations and horizontally mounted 5" plunge plates accept a maximum monitor weight of 18 kg / 39.7 lbs. An additional 4 kg / 8.8 lbs may be mounted on a down post underneath the front swivel part. Depending on material and tensile factor all weight indications have a 4-fold and/ or 6-fold safety factor according to IEC 60601-1 standards.

Custom-designed solutions and MRI applications

Please contact us for custom-designed solutions.

Product marking


All products which are classified as medical product risk class 1 are CE marked. As **medical product risk class 1** they are conform with the regulations (EU) 2017/745 (MDR) and IEC 60601-1. Other standards of the IEC 60601 series are not applicable to our products, regardless of their version.

MD A stand-alone product that is already classified as a medical product class 1 before being integrated into an overall medical device. The product is CE-marked. A declaration of conformity is available.

Accessories or spare parts that have no independent function and are therefore not subject to the MDR. These products are not CE-marked. A declaration of conformity is not required.

Warranty

All CIM products have a 5-year warranty to be free of defects in material and fuctionality from the date of delivery.

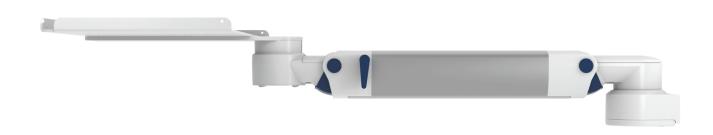



How to create part numbers

Part numbers can be easily created by following the steps below:

Please replace the letters in green with the part no. code as per your requirements.

1. Components' grounding:

 $\mathsf{A} \qquad \qquad = \qquad \mathsf{grounded}$

2. Choose the arm version required:

224E / 230E = support arm pivot

33030E = extended pivot arm light duty 431E = height adjustable arm

73130E = extended height adjustable arm

3. Specify where you want to mount the arm and select the fixing (Y):

2F = horizontal rails 10 x 25, 8 x 35, 10 x 30, 10 x 50 mm

3 = vertical rail 10 x 25 mm 4S = vertical wall channel

9 = Löwenstein Medical / ITD / mounting channel

13 = Mindray medical supply units 17 = Amico pendants & headwalls

4. Choose the colour requested for decorative parts (C):

 $m gr = RAL \, 7024 \, (graphite \, grey)$ $m bl = RAL \, 5013 \, (cobalt \, blue)$









How to create part numbers

5. Choose the colour requested for the aluminium die-cast parts (D):

1 = RAL 9016 (traffic white) 2 = RAL 9002 (grey white)

6. Specify the spring depending on the weight of your monitor/ device - only with height adjustable arms (X):

 $= 0 - 4 \, \text{kg} / 0 - 8.8 \, \text{lbs}$

7. Specify the requested size of the keyboard tray and mouse (T):

<u>Standard</u>	Magnetic	With wrist rest		Dimensions
30	30S		=	300 x 210 mm / 11.8 x 8.3"
40	40S		=	400 x 150 mm / 15.7 x 5.9"
50	50\$	50W	=	500 x 200 mm / 19.7 x 7.9"
58	58\$	58W	=	580 x 200 mm / 22.8 x 7.9"
65	65S	65W	=	650 x 200 mm / 25.6 x 7.9"
RC			=	480 x 220 mm (retractable mouse tray left/right 226 mm / 8.9")

Example

01 Grounding		02 Arm version		03 Fixing	04 & 05 Colour		06 Spring tension		07 Keyboard tray
Α	_	431E	_	Υ	CD	_	60	_	T
Α	-	431E	-	48	gr1	_	60	_	58\$



Product highlights

Non-height adjustable arm

Pivot and extended pivot arms offer full cable integration and easy cleaning for infection control in all critical care areas in the clinical environment. Our 5-years warranty guarantees reliable, trouble-free and maintenance-free operation. The swivel function at the fixing point as well as the tilt/swivel function of the monitor adaptation offers optimum positioning of your device. The extended pivot arm is a two-part arm with central pivot point between the two arm sections. All bottommounted monitors have the safety feature of a 20° tilt stop.

Height adjustable arm

The height adjustable arm offers **full cable integration**, one-wipe clean surfaces and our **5-year warranty** on the entirety of the product. Vertically and horizontally adjustable, the arm offers optimal positioning of your device. A constant viewing angle when vertically adjusted is guaranteed by a parallel linkage. **Weight-optimized springs counter-balance** the monitor without making any adjustments. All bottom-mounted monitors have the safety feature of a **20° tilt stop**. Individual rotational stops may be integrated in any joint.

To provide the perfect combination of arm and monitor, the CIM articulated height adjustable arm has a simple quarter-turn lock to hold the arm safely in position when removing the monitor. Furthermore the arm is equipped with the unique safety feature of a slow-release gas spring. Should anybody try to remove the monitor before locking the arm in position, the arm will only rise at a slow and even tempo.





Support arm pivot





Keyboard tray T

Colours to choose D

Max. load capacity support arm
Max. load keyboard tray

Product weight

Part number

dimensions page 15 - 18

aluminum parts

RAL 9016 traffic white / RAL 9002 grey white

16 kg / 35.3 lbs

5 kg / 11 lbs

arm type 240: 1.8 - 3.4 kg / 9.5": 4.0 - 7.5 lbs (according to dimensions of keyboard tray)

arm type 300: 1.9 - 3.5 kg / 11.8": 4.2 - 7.7 lbs (according to dimensions of keyboard tray)

arm type 240

arm type 300

A-224E-Y.D_T

details page 19

A-230E-Y.D T

Fixings Y



Adapter for vertical wall channel



Adapter for Mindray medical supply units

2

Horizontal rail clamp



Adapter for Amico pendants & headwalls

Vertical rail clamp



Adapter for Löwenstein Medical / ITD / mounting channel



Universal pole clamp



Electrical control

Fixed pole clamp







Support arm pivot

Technical data - arm type 300, keyboard tray 500 x 200 mm / 19.7 x 7.9"

SIDE VIEW

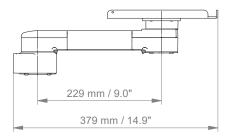

TOP VIEW

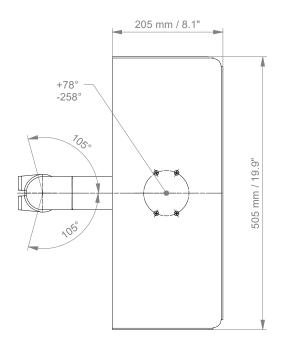






Extended pivot arm light duty



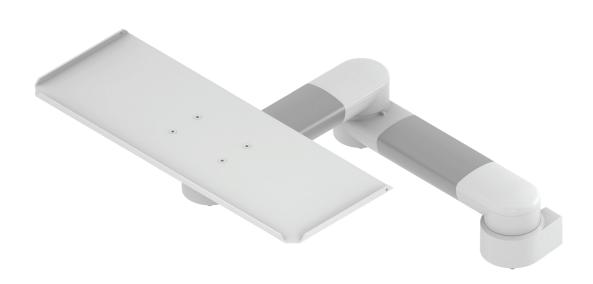

Keyboard tray T

Colours to choose D

Max. load capacity support arm Max. load keyboard tray

Product weight

Part number

dimensions page 15 - 18

aluminium parts

RAL 9016 traffic white / RAL 9002 grey white

6 kg / 13.3 lbs

5 kg / 11 lbs

2.8 - 4.4 kg / 9.5": 6.2 - 9.7 lbs (according to dimensions of keyboard tray)

A-33030E-Y.D_T

Fixings Y



Adapter for vertical wall channel



Adapter for Mindray medical supply units

2F

Horizontal rail clamp



Adapter for Amico pendants & headwalls

details page 19



Vertical rail clamp



Universal pole clamp



Fixed pole clamp

Adapter for Löwenstein Medical / ITD / mounting channel







Extended pivot arm light duty

Technical data - keyboard tray 500 x 200 mm / 19.7 x 7.9"

SIDE VIEW

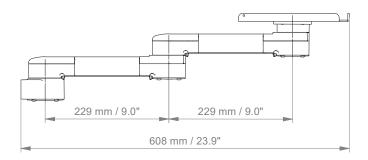

TOP VIEW

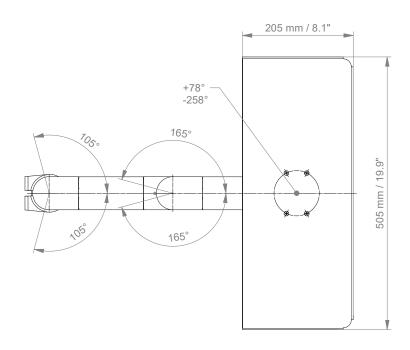



Height adjustable arm





Keyboard tray T

Height adjustment

Colours to choose CD

Max. load capacity support arm Max. load keyboard tray

Product weight

Balancing range

Part number

dimensions page 15 - 18

slow-flowing spring, parallel linkage for a constant viewing angle

decorative parts RAL 5013 cobalt blue / RAL 7024 graphite grey aluminum parts RAL 9016 traffic white / RAL 9002 grey white

20 kg / 44.1 lbs 5 kg / 11 lbs

4.4 - 6.0 kg / 9.7 - 13.2 lbs (according to dimensions of keyboard tray)

600 N (0 - 4 kg / 0 - 8.8 lbs)

A-431E-Y.CD_60_T

Fixings Y



Adapter for vertical wall channel



Adapter for Mindray medical supply units

2F

Horizontal rail clamp



Adapter for Amico pendants & headwalls

details page 19



Vertical rail clamp



Universal pole clamp



Fixed pole clamp

Adapter for Löwenstein Medical / ITD / mounting channel







Height adjustable arm

Technical data - keyboard tray 500 x 200 mm / 19.7 x 7.9"

SIDE VIEW

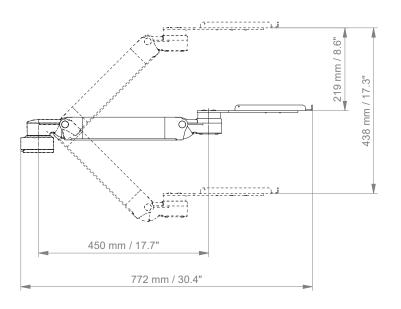

TOP VIEW

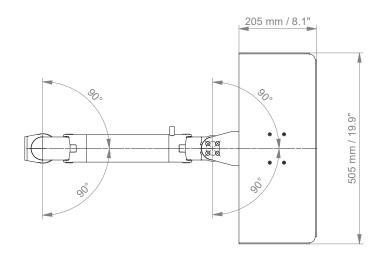



Extended height adjustable arm



Keyboard tray T

Height adjustment

Colours to choose CD

Max. load capacity support arm Max. load keyboard tray

Product weight

Balancing range

Part number

dimensions page 15 - 18

slow-flowing spring, parallel linkage for a constant viewing angle

decorative parts RAL 5013 cobalt blue / RAL 7024 graphite grey aluminum parts RAL 9016 traffic white / RAL 9002 grey white

20 kg / 44.1 lbs 5 kg / 11 lbs

5.5 - 7.1 kg / 12.1 - 15.7 lbs (according to dimensions of keyboard tray)

600 N (0 - 4 kg / 0 - 8.8 lbs)

A-73130E-Y.CD_60_T

Fixings Y



Adapter for vertical wall channel



Adapter for Mindray medical supply units

2F

Horizontal rail clamp



Adapter for Amico pendants & headwalls

details page 19



Vertical rail clamp



Universal pole clamp



Fixed pole clamp


Adapter for Löwenstein Medical / ITD / mounting channel




Extended height adjustable arm

Technical data - keyboard tray 500 x 200 mm / 19.7 x 7.9"

SIDE VIEW

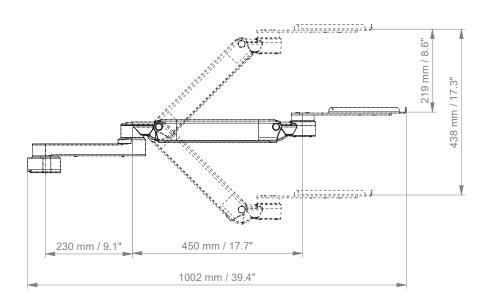

TOP VIEW

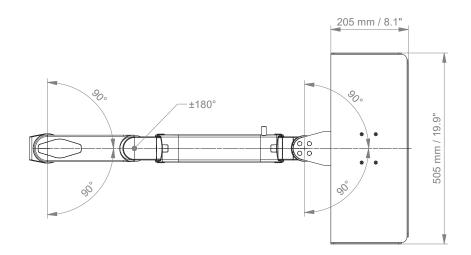






Keyboard tray options T


Keyboard tray 300 x 210 mm / 11.8" x 8.3"



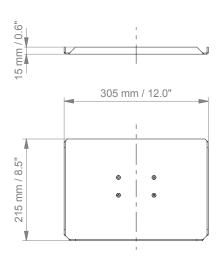


Keyboard tray 400 x 150 mm / 15.7" x 5.9"



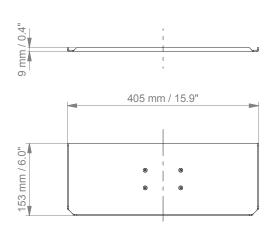

Keyboard tray 500 x 200 mm / 19.7" x 7.9"



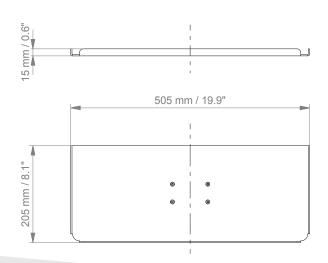






Keyboard tray 580 x 200 mm / 22.8" x 7.9"




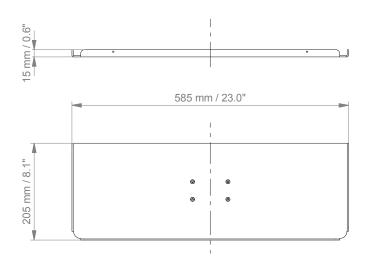

Keyboard tray 650 x 200 mm / 25.6" x 7.9"






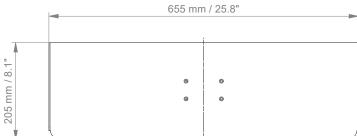






Keyboard tray with retractable left / right mouse tray 480 x 220 mm / 18.9" x 8.7", mouse tray 226 mm / 8.9"

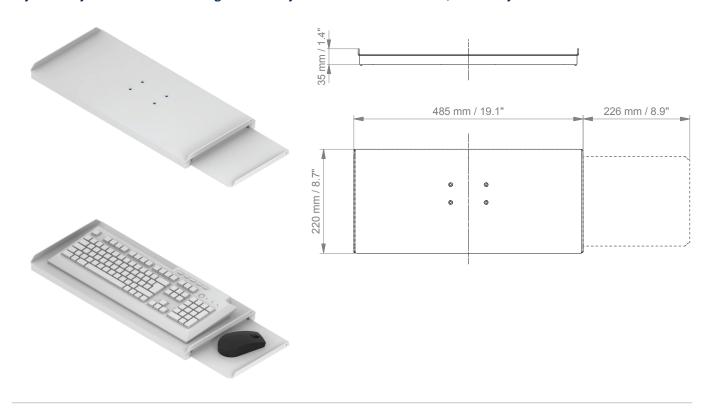


Keyboard tray with wrist rest 500 x 200 mm / 19.7×7.9 "

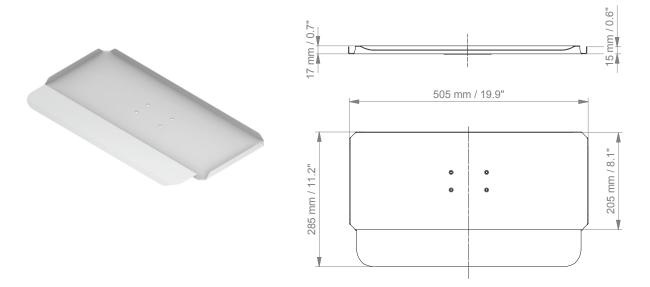



Keyboard tray options T

Keyboard tray with wrist rest 580 x 200 mm / 22.8 x 7.9"

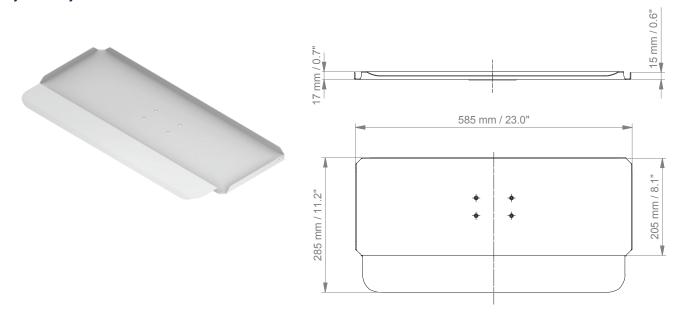





Fixings Y



Illustration	Description	Notes	Weight	Part no.
	Adapter for vertical wall channel	To mount to CIM wall channel (compatible with GCX channel) Please order separately: CIM wall channel, see catalogue accessoires	0.6 kg / 1.3 lbs	4\$
: : ;	Horizontal rail clamp	To mount to horizontal European rails 10 x 25, 8 x 35, 10 x 30, 10 x 50 mm	1.1 kg / 2.4 lbs	2F
	Vertical rail clamp	To mount to vertical European rail 10 x 25 mm	0.5 kg / 1.1 lbs	3
100	Universal pole clamp	V-block to mount to pole Ø 23 - 40 mm / 0.9 - 1.6"	0.6 kg / 1.3 lbs	1V
	Fixed pole clamp	To mount to pole ø 25 mm / 1.0" To mount to pole ø 30 mm / 1.2" To mount to pole ø 35 mm / 1.4" To mount to pole ø 38 mm / 1.5"	0.3 kg / 0.7 lbs 0.4 kg / 0.9 lbs 0.4 kg / 0.9 lbs 0.4 kg / 0.9 lbs	1 1B 1F 1C
6 6 6 9 6 6 6 9	Adapter medical supply units	For Mindray medical supply units	0.8 kg / 1.8 lbs	13
+ 00	Adapter pendants & headwalls	For Amico pendants & headwalls	0.6 kg / 1.3 lbs	17
6 6	Adapter pendants & headwalls	For Löwenstein Medical / ITD / mounting channels	0.4 kg / 0.9 lbs	9



CIM med GmbH

Euro-Industriepark Margot-Kalinke-Str. 9 80939 Munich / Germany

T +49 89. 978 94 08-0

CIM med Inc.

2024 E Independence Commerce Dr Matthews, NC 28105 / USA

T +1 855 895 4722

info@cim-med.com www.cim-med.com

